

Catheter for laparoscopy – Rif. G2-LPC / G2-LPC-RIG

TECHNICAL CARD

DESIGNATION

Catheter for laparoscopy

DESCRIPTION

Device consisting of:

- Hollow catheter with luer tip, for the exclusive application of GLUBRAN 2 surgical glue;
- (Flexible or rigid) Sheath

INSTRUCTIONS

Single use, exclusively for application of GLUBRAN 2 surgical glue during laparoscopic surgery. The device can be used only with 5 mm diameter trocar. Insert the entire device into the trocar, controlling compatibility and seal. Then connect the syringe of GLUBRAN 2 surgical glue to the catheter luer lock. Apply the glue drop by drop on the area to be treated, and to avoid the tip to be clogged up, avoid the catheter tip gets in contact with the area. When the procedure is completed, remove first the catheter and then the guide.

MATERIALS

Polyethylene, eraclene, polyester PEBAX7233SA01, TERLUX 2802 TR, Stainless steel

CODE

G2-LPC

G2-LPC-RIG

STERILIZATION

Gamma rays

SHELF LIFE

5 years

PACKAGING

Double sterile peel pouch

Any reuse of the disposable product involves risk of infection to the patient for a potential contamination of the product itself and the harm due to a possible impairment in its effectiveness

STORAGE

Standard – Keep product in its original packaging in a clean, cool, dry place at temperature below 30°C

COMPLIANCE DIRECTIVE

MMD/93/42/EEC and subsequent updating

QUALITY CONTROL

During production and on the biocompatibility of the materials

CLASS

Ila